



“  
I’m taking the  
work out of  
denture work  
”




**Southern  
Cross Dental**  
A Modern Dental Pacific Company

**Digital Dentures Clinical Protocol**

## Full Dentures

This quick reference provides a basic guide to the clinical protocol of digital full dentures, ensuring accurate and comfortable outcomes.

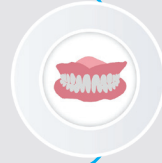

### For patients **with existing Full Dentures (Upper/lower/ both)**


### For patients **without existing Full Dentures**

- 1 Reline close mouth impression taken in centric relation.
- 2 Scan denture fitting surface, borders, buccal surface including teeth and occlusal of teeth (palatal/lingual areas not necessary) all at once.
- 3 Take opposing scan. If opposing is also a full denture same as steps 1 and 2.
- 4 Left and right bite scans taken in centric relation intraorally. Do not scan bite outside the mouth. Do not use bite material. If vertical dimension needs to be increased, place occlusal pivots (composite or wax) and scan left and right bites intraorally in centric relation.
- 5 **Option 1:** Send to us to fabricate MMRs on a hard base and when received back from us, apply 'patient with existing protocol'.  
OR  
**Option 2:** Intraoral scan of maxillomandibular relation. Leaf gauge can be used for approximate vertical dimension and maxillomandibular relation.
- 6 Monoblock try-in will be produced by the lab and sent to the clinic.
- 6 Once monoblock printed try-in is received, follow protocol for patient with existing full dentures.

**Milled dentures fabricated and returned for final issue.**


## Partial Dentures

This quick reference provides a basic guide to the clinical protocol of digital partial dentures, ensuring accurate and comfortable outcomes.

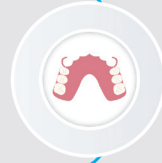

### For patients **with existing** Partial Dentures

### For patients **without existing** Partial Dentures

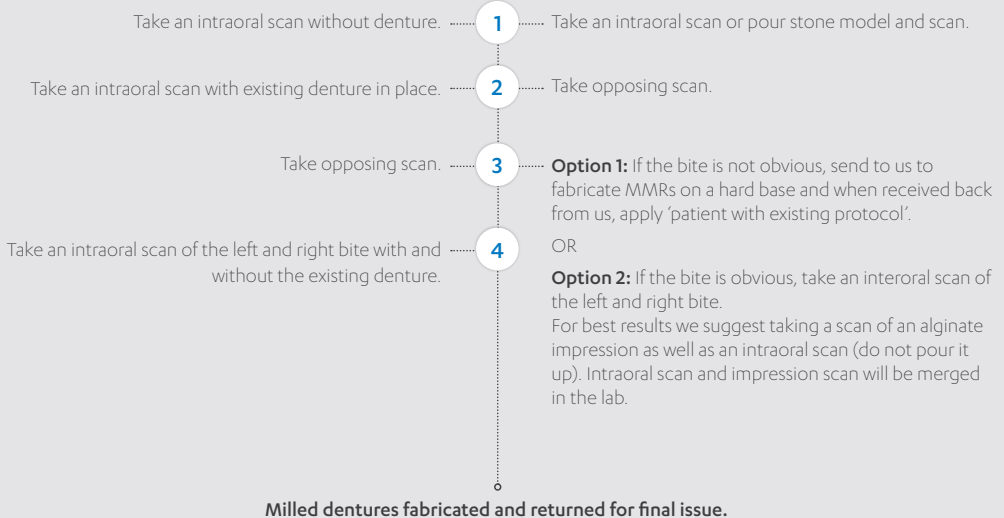

02 8062 9800



[scdlab.com](http://scdlab.com)